

FRD Competency ✓

Pink Slip / Pink Slip Elite / Elite Slip Tube



Name:

Title:

Unit:

Date:

Upon completion of this product orientation the caregiver will properly return-demonstrate:

BOOSTING		Meets Performance Standards	Requires Review
1	Positions bed or stretcher at waist level.		
2	Opens, unfolds and separates the FRD.		
3	Properly rolls the FRD for placement.		
4	Places the FRD under the patient's head/pillow correctly.		
5	Unrolls the FRD under itself (and under draw sheet).		
6	Carefully glides patient up in bed using the draw sheet.		
7	Removes the FRD by unrolling it under itself.		
8	Tucks the FRD securely at the foot of the bed.		

LATERAL TRANSFER		Meets Performance Standards	Requires Review
1	Positions bed or stretcher at waist level.		
2	Opens, unfolds and separates the FRD.		
3	Properly folds the FRD in preparation for placement.		
4	Correctly positions the FRD under the patient.		
5	Second surface positioned with no more than 3"-4" gap, lowered 2"-3" and brakes locked.		
6	Caregivers demonstrate proper transfer technique of patient.		
7	Removes the FRD by unrolling it under itself.		
8	Tucks the FRD securely at the foot of the bed.		