

# SlingSupply™

## Sling Basics

SlingSupply \* Clackamas, OR \* 800-255-3126 \* [www.slingsupply.com](http://www.slingsupply.com)

Re: SlingSupply Patient Slings, excluding disposable slings

### **Warranty:**

All slings are warranted for 3 months from their delivery date. This warranty is not linked to the number of laundering cycles.

### **Construction:**

Covering: 100% Dupont 1000d Cordura ® nylon with urethane coating.

Foam: closed cell cross-linked polyolefin.

Strapping: Polypropylene filament yarn containing organic lubricant.

Stitching thread: bonded polyester

### **Laundering:**

Water temperature 140°F to 212°F


Mild detergent

Tumble dry at warm temperature

No Chlorine oxidizer

### **Inspections:**

Slings should be inspected prior to each use for any signs of wear or damage at the inspection points shown below and replace as needed.

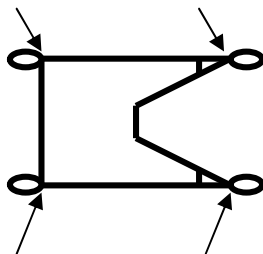

### **Instructions on Use:**

- Place the sling under the patient by using a rolling technique, if patient is on the floor or bed
- If patient is sitting up in a chair, slide the sling down the patient's back form behind the patient's head
- Bring the leg bands under and up between the thighs
- Route leg bands through each other and connect to opposite ends of lifting bar
- Connect loops at head end of sling to the same side of lifting bar