

Totalift Competency ✓

Totalift Transfer Chair



Name:

Title:

Unit:

Date:

Upon completion of this product orientation the caregiver will properly return-demonstrate:

PERFORMANCE STANDARDS: EQUIPMENT CAPABILITY AND SAFETY REVIEW		Meets Performance Standards	Requires Review
1	The appropriate applications for the Totalift chair		
2	Totalift's maximum weight capacity		
3	All functions of Totalift's features and accessories		
4	Proper position of "Brake/Steer" pedals for patient transfer & transport		
5	Leg section and footrest adjustment		
6	Side rail engage & release mechanism		
7	The process for raising and lowering the Totalift		
8	The process for reclining Totalift		
9	The process for performing a patient transfer using Totalift's lateral transfer mechanism		
10	Totalift accessory application (patient tray, headrest, upper side rails)		

PERFORMANCE STANDARDS: LATERAL TRANSFER MANEUVERS AND POSITIONING		Meets Performance Standards	Requires Review
1	Accurately assess patient mobility level and appropriate application of the device		
2	Clearly communicate the transfer process to the patient		
3	Position the Totalift adjacent to the bed with patient surface flat, and brakes engaged		
4	Adjust the height of the Totalift to a position that is appropriate for the caregiver		
5	Effectively & safely execute the patient transfer procedure		
6	Adjust Totalift's height, back & leg sections, and foot rest for patient comfort		
7	Set the Brake/Steer pedal assembly in the proper position for transport		